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Ice Grabber RFM with 5 Spikes

Read the instruction manual before using Walking frame RFM foldable for the first time and keep it safe for later use.

Intended purpose

The ice grabber is suitable for mounting to the walking frame so that it can also be used outside on icy ground as part of a rehabilitation measure after a hip or knee operation.

Indication

For illnesses or effects from injuries involving

- disorders of the musculoskeletal system and at the same time coordination disorders or disturbance of equilibrium,
- for preserving the remaining walking functions and if a walking exercise needs to be performed at the same time
- loss of strength

Mounting/Use

Unscrew the two screws from the threads of the bracket and put them aside. Place the holder on the pole/frame with the rubber pads between the pole and the holder to avoid scratches.

In the lower position, the foot of the ice grabber must be positioned just below the cane cap. Push the holder accordingly far up at the pipe of the walking frame. Turn in the two screws and tighten them firmly.

For use on ice and snow lock the Ice Grabber in the down position. On firm ground (cement etc.) or in buildings you should fold it up in order to avoid damages to the Ice Grabber or the flooring!

Safety Instructions

1. Ensure that the Ice Grabber is securely fixed before using it.
2. Ensure not to endanger/hurt any other people with the Ice Grabber. The spikes are quite sturdy and sharp!
3. Inside buildings use the Ice Grabber only in the upper position. The spikes can damage the flooring otherwise.
4. Do not use a defective Ice Grabber – there is danger of falling!

Cleaning/Storage

Clean the Ice Grabber with mild soapy water and a dry cloth. Do not use caustic or abrasive cleaning agents! Store the ice grabber dry in the packaging provided.

Contraindication

The use of walking frame is not indicated for persons with perceptual disorders, severe balance disorders, amputated limbs or severe pain.

Material/Size

Stainless steel, nickel-plated; for mounting on walking aids with Ø16-22 mm

Length approx. 10 cm

Rehaforum Medical GmbH
Daimlerstr. 12a
25337 Elmshorn
www.rehaforum.com

Art.80000844

4 0 0 4 1 3 1 4 0 2 0 1 5 1