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1 General instructions

1.1 Introduction

These operating instructions contain information on how to use the raised toilet seat safely and properly.

Read the operating instructions thoroughly prior to the use of the raised toilet seat and follow all instructions.

Keep all documents in a safe place.

1.2 Proper use

The raised toilet seat is used exclusively to raise the toilet seat and is therefore intended to be fitted to a toilet bowl.

Any other form of use is not permitted.

1.3 Disposal

Old appliances and packaging must be disposed of and recycled in accordance with the valid national regulations.

1.4 CE mark

The product complies with the medical products directive 93/42/EEC.

1.5 Reuse

The product is suitable for reuse. The amount of reuse depends on the condition of the product in question.

Prior to reuse, follow the manufacturer's reconditioning and hygiene guidelines, and check the technical and functional safety of the product as well.

1.6 Service life

The product is designed for a service life of at least 5 years, provided that it is used in accordance with its intended use and that all cleaning and safety instructions are observed.

This service life can be exceeded if the product is carefully handled, cared for, and used.

2 Safety instructions

Do not overload the raised toilet seat (max. load: 130 kg).

In the case of a wall-mounted toilet bowl, do not exceed the permissible load weight of the toilet bowl.

Make sure that the raised toilet seat and buttocks are dry before going to the toilet.

In the case of the model with armrests: distribute the weight evenly on armrests. Do not load armrests with the entire body weight.

Do not use the raised toilet seat if the seat support is damaged.

Make sure that the front height adjustment rod has engaged and that the rear height adjustment rod is fastened with the screws.

Make sure that the raised toilet seat is fastened to the toilet bowl by the screws.

Do not expose raised toilet seat to temperatures above +85 °C.

Check the product periodically for damage and safe assembly.

3 Description

The raised toilet seat is available in the following versions:

- With armrest (Fig. 1)
- Without armrest (Fig. 2)

3.1 Technical data

Refer to "Technical data" at the end of the operating instructions.

3.2 Adjusting the seat height

1. Depending on the desired seat height, install front height adjustment rod in position A, B, or C (refer to Fig. 3, 4, 5).
2. Release 4 screws at the rear height adjustment rod (refer to Fig. 6).
3. Insert the rear height adjustment rod consistently (refer to Fig. 6).
4. Tighten 4 screws by hand (refer to Fig. 6).

3.3 Attaching the raised toilet seat

1. Move rods for quick lock (1, Fig. 7) into right position, such that the rods can be guided through the holes in the toilet bowl.
2. Put the raised toilet seat onto the toilet bowl. While doing this, guide the rods through the holes.
3. Compress bottom end of the quick locks (2, Fig. 7) and push them onto the rods until they are in contact with the toilet bowl.
4. Release the bottom end of the quick lock, push the quick lock upwards and, then, tighten it by hand (Fig. 7).
5. In the case of the model with armrests: push armrests onto the holding pipes. Make sure that the spring catches engage.

3.4 Removing the raised toilet seat

1. Release quick lock and, then, compress the top end and pull quick lock downwards.
2. Pull quick lock off from the rod and remove raised toilet seat.

3.5 Removing toilet seat and lid

1. Fold toilet seat and lid upwards to 45° and pull off evenly upwards.

3.6 Removing the armrests

1. Fold armrests upwards to 90°.
The armrest unlock automatically.
2. Pull armrest off from the holding pipe.

4 Maintenance and care

The product can be cleaned and disinfected with customary agents. Do not use any cleaning agents containing solvents.

Use disinfectant if necessary, as well as in the event of application for more than one person. Follow the disinfectant manufacturer's instructions.

When adhering to the specified cleaning and safety instructions, the product is maintenance-free.

After cleaning, dry off the raised toilet seat with a clean cloth.

Have any necessary repairs done by an authorised specialist workshop.

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