drive

DeVilbiss
HEALTHCARE

GoLite 200 Rollator

CE

Gebrauchsanweisung – Operating instructions – Manuel d'utilisation – Gebruiksaanwijzing

Modell | Model
Modèle | Model

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Indication / Contraindication

This walking aid is suitable for a single user with limited mobility. It is suitable for indoor and outdoor use.

The purpose of walking devices is to compensate for or at least improve restricted walking ability in users (expansion of range and/or mobility).

Persons with restricted hand/arm functions require special adjustments/versions of handgrips or hand rests/armrests.

For persons with balance disorders are walking frames not recommended, this is to be checked by an experiment in an individual case. If it is necessary a companion must be present.

Scope of delivery contents

1. Rollator GoLite 200
2. Basket
3. Back Rest
4. Cane Holder
5. Operating instructions

Unfolding

Position the Rollator on its wheels. Push the front frame forward by pressing down on the rollator until it stands on four wheels.

Then press the seat down until (Picture 1).

Assembly Back Rest

Insert the back rest from the top into the reception tubes until the push buttons lock in position.

Fitting the Basket

Hook in the basket with the hooks (see circle picture 3) at the frame under the seat plate and then pivot it down until the basket attaches the pipe.

Height adjustment handgrips

Insert the handle tubing into the receiver until the desired height is reached. Line up the holes on the tubing with the receiver and secure using the height adjustment screw.

Breaking / Parking

Pull both brake levers upwards to apply the brakes whilst the rollator is moving (see picture 5). Always brake slowly and steady.

To apply the parking brakes, ensure the Rollator is stationary and pull down both brake levers until they lock into position (see picture 6).

To release the parking brakes pull up on both brake levers until the brake is released.

The brakes performance can be affected by abrasion of the wheels. **Therefore test regularly the brakes.**

Adjustment of brakes

For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture 7) or with the suitable tool (not included) on the brake re (picture 8).

Sitting

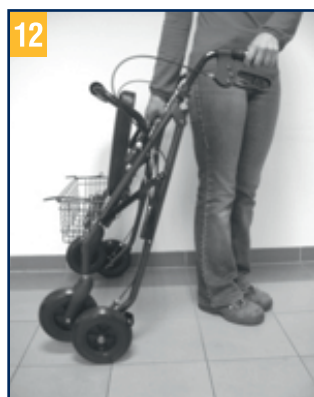
- Position the rollator on a flat and solid surface.
- Then lock the brakes (parking position).
- The rollator must stand and must not slip.
- Sit straight up and do not lean backwards (see picture 9).

Folding / Transport / Storage

To fold the Rollator first flip the seat upwards.

Then pull up on the chrome centre bar of the rollator. The rollator will fold up and can be move like a trolley or be stowed away (picture 10-12).

Reprocessing

The product is suitable for reprocessing in line with manufacturer's recommendations.

Safety instructions

- Use only as a walking aid!
- Use only on even and solid surfaces.
- The rollator must not be used for standing on or as a wheelchair! It must not be used for transport of persons or animals.
- Maximum user weight: 200 kg
- The use on escalator`s is expressly forbidden!
- Use basket only for transport of objects and do not overload. Maximum load: 5 kg
- Do not expose the product to temperatures above 40°C for a long time, as this can cause the rollator parts to overheat and burns to the user.
- The brakes performance can be affected by abrasion of the wheels. Therefore test regularly the brakes.

Cleaning / Maintenance

- Clean regularly by hand with a damp cloth!
- Do not use any aggressive detergents!
- Regularly check the screws and fasteners on the rollator and retighten if necessary.

Warranty

This product is guaranteed for a period of 24 months from date of delivery. Covers all defects in the product that are shown to be caused by material or production defects.

Damage caused by wear to items such as wheels, handles and brakes is not covered by the guarantee.

Damage that arises due to abnormal use is excluded.

Subject to technical alterations. Errors and omissions excepted.

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